



EXPRESS WIPES GERM AWAY

DESCRIPTION

EXPRESS WIPES GERM AWAY wipe is a non-woven disposable cloth containing a stable, low pH formulated disinfectant and deodorant for use on hard, non-porous surfaces where control of cross-contamination is required. *Note: May be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection. Use on food contact surfaces must be followed by a potable water rinse. Effective against the following pathogens in 2 minutes at 20°C/68°F, on hard, non-porous surfaces when used as directed: *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Salmonella enterica*, *Escherichia coli* 0157:H7, *Mycobacterium tuberculosis*, *Methicillin Resistant Staphylococcus aureus (MRSA)*, *Vancomycin Resistant Enterococcus faecalis (VRE)*, *Human Hepatitis B virus, *Human Hepatitis C virus, *Herpes simplex virus 2 (genital Herpes virus), *Influenza A (A2 Japan). *HIV-1 (AIDS virus) ** (At an exposure time of 1 min.), **See label for complete HIV-1 (AIDS virus) information.

APPLICATIONS

Areas of use: hospitals, intensive care units, surgery, recovery, anesthesia, X-ray, Cat. Lab., orthopedics, newborn nursery, respiratory therapy, emergency medical settings, laboratories, clinics, nursing facilities, medical offices, dental offices, veterinary facilities, janitorial, kitchens, musty closets, sickrooms, bathrooms. For use on hard non-porous surfaces only, such as stainless steel, Formica, glass tables, carts, baskets, counters, cabinets, telephones.

FEATURES & BENEFITS

- Can be used anywhere on nearly any hard, nonporous surface to disinfect and deodorize
- Safely decreases chances of cross-contamination
- Dispenses one at a time so you only use what you need for less waste

SPECIFICATIONS

FORM: Premoistened Towel
ODOR: Citrus
COLOR: White Cloth
DETERGENCY: N/A
TOXICITY: N/A
WETTING ABILITY: Excellent
PHOSPHATES: N/A
STORAGE STABILITY: 1 year +
COLD STABILITY: 32°F

FLASH POINT: N/A
BOILING POINT: N/A
SOLUBILITY IN WATER: 100%
SPECIFIC GRAVITY: 1.00
% VOLATILE BY VOLUME: 100%
EVAPORATION RATE: N/A
BIODEGRADABLE: Yes
FLAME EXTENSION: N/A
pH: N/A

ACTIVE INGREDIENTS:
 N-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) Dimethyl Benzyl Ammonium Chloride 0.12%
 N-Alkyl (68% C12, 32% C14) Dimethyl Ethyl Benzyl Ammonium Chloride 0.12%
 Isopropyl Alcohol 41.58%
OTHER INGREDIENTS +: 58.18%
TOTAL: 100.00%
 + Does not include weight of dry wipe.

LABEL INFORMATION

EXPRESS WIPES GERM AWAY

A non-woven disposable cloth containing a stable, low pH formulated disinfectant and deodorant for use on hard, non-porous surfaces where control of cross-contamination is required. Areas of use: hospitals, intensive care units, surgery, recovery, anesthesia, X-ray, Cat. Lab., orthopedics, newborn nursery, respiratory therapy, emergency medical settings, laboratories, clinics, nursing facilities, medical offices, dental offices, veterinary facilities, janitorial, kitchens, musty closets, sickrooms, bathrooms. For use on hard non-porous surfaces only, such as stainless steel, Formica, glass tables, carts, baskets, counters, cabinets, telephones.

PREPARATION OF DISPENSER: Remove cover from canister and discard covering seal. From center of wipes roll, pull up a wipe corner, twist into a point and thread through the tear throat in the underside of the lid cover. Pull wipe through about one (1) inch. Replace the lid cover on container, assuring a secure fit. Pull out first wipe and snap at a 90° angle. Remaining wipes feed automatically for next use. Always reseal hinged cap after each use to insure moisture retention.

DIRECTIONS FOR USE: It is a violation of U.S. Federal Law to use this product in a manner inconsistent with its labeling. This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be

used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection. This product is not for use on the inhalation therapy and/or pulmonary diagnostic equipment.

DISINFECTION: Thoroughly wet pre-cleaned, hard, non-porous surface with a wipe, keep wet for 2 minutes (5 minutes if fungus is suspected), and allow to air dry. Use as many wipes as needed for the treated surface to remain wet for the entire contact time. For use as a general disinfectant wipe on hard, non-porous, inanimate surfaces such as chairs, baskets, counter tops, exam tables, cages, toilets, basins, tubs, bed springs, frames, chart racks and equipment, cabinets, door knobs, sinks, mirrors, light fixtures and switches, supply and trash carts, telephones, waste cans and laundry hampers, wheel chairs, and walkers, urinals, plastic and other similar hard surfaces. You may use a wipe to pre-clean the surface to be disinfected, but thorough re-wiping with a fresh wipe is required for disinfection. Use on food contact surfaces must be followed by a potable water rinse.

This product is effective against the following pathogens in 2 minutes at 20°C/68°F, on hard, non-porous surfaces when used as directed: *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Salmonella enterica*, *Escherichia coli* 0157:H7, *Mycobacterium tuberculosis*, *Methicillin Resistant Staphylococcus aureus (MRSA)*, *Vancomycin Resistant Enterococcus faecalis (VRE)*, *Human Hepatitis B virus, *Human Hepatitis C virus, *Herpes simplex virus 2 (genital Herpes virus), *Influenza A (A2 Japan). *HIV-1 (AIDS virus) ** (At an exposure

time of 1 min.), **See back panel for complete HIV-1 (AIDS virus) information.

This product is effective against the the following pathogens in 5 minutes at 20°C/68°F, on hard, non-porous surfaces when used as directed: *Trichophyton mentagrophytes* (Athlete's foot fungus).

DEODORIZING: Wipe hard, non-porous surface thoroughly to eliminate odors from bathrooms, fecal matter, urine, burns, smoke, mildew and mold, decay, stale air, tobacco, sickrooms and musty closets.

CLEANING: To clean surfaces, wipe thoroughly with a fresh wipe, which may be used to deodorize at the same time as cleaning. When used to clean a hard non-porous surface prior to disinfection, a second wipe must be used for disinfection.

THE ACTIVITY OF EXPRESS WIPES GERM AWAY AGAINST THE HIV-1 (AIDS VIRUS):

EXPRESS WIPES GERM AWAY completely inactivates the HIV-1 (AIDS virus) on hard, non-porous surfaces when used at a contact time of 1 minute at room temperature (20°-25°C). EXPRESS WIPES GERM AWAY kills HIV-1 on precleaned environmental surfaces/objects soiled with blood/body fluids, in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type-1 (HIV-1) (associated with AIDS).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS:

Personal Protection: Specific barrier protection items to be used when handling items soiled with blood or body fluids are disposable latex gloves, gowns, masks and eye coverings. Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of EXPRESS WIPES GERM AWAY. Contact Time: HIV-1 virus is inactivated in 1 minute,

however, other pathogens require 5 minutes exposure for complete kill. See disinfection directions. Disposal of Infectious Materials: Materials that may contain blood or other body fluids should be autoclaved and disposed of according to federal, state and local regulations for infectious waste disposal.

STORAGE AND DISPOSAL:

Store in cool area. Keep away from heat or open flame. Do not reuse wipes. Dispose of used wipe in trash. Container disposal: Non-refillable container. Do not reuse or refill this container. Offer for recycling if available.

PRECAUTIONARY STATEMENTS:

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

WARNING: Causes substantial, but temporary eye injury. Do not get in eyes or on clothing. Wear safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco. Wear disposable latex gloves, gowns, masks and eye coverings, when handling HIV-1 infected blood or body fluids. Avoid contamination of food. Remove and wash contaminated clothing before reuse.

FIRST AID:

If in eyes: Hold eye open and rinse slowly and gently with water for at least 15-20 minutes. Remove contact lenses, if present, after first 5 minutes, then, continue rinsing eye. Call a poison control center or doctor for treatment advice. **If on skin or clothing:** Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice. Wash contaminated clothing before reuse. **WASH THOROUGHLY AFTER HANDLING WIPES.**

For medical emergencies or pesticide incidents the poison control center can be called at 1 888 740 8712

EPA Reg. No. 59894-10-44446 EPA Est. No.44446-TX-1

Made in USA

HEALTH	1
FLAMMABILITY	1
REACTIVITY	0
PERSONAL	A



PACKAGING INFORMATION

50 premoistened towels per container, 6 containers per case.